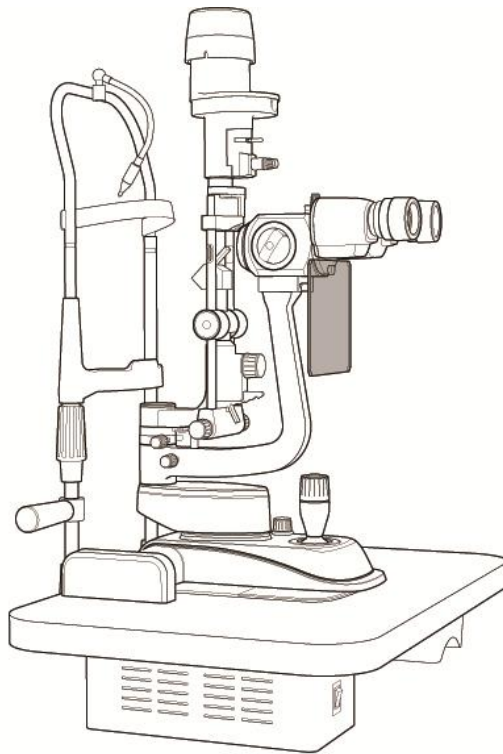
# ENG

# Operator's Manual

---

SLIT LAMP MICROSCOPE HS-7000

## IMPORTANT NOTICE

### WARNING

Potential electromagnetic or other interference between medical equipments and other devices being operated together in the same environmental may expert an adverse influence on functioning of the medical equipment. Non-medical equipments not in compliance with the requirements of EN 60601-1 and EN 60601-1-2 should not be used together in the same environmental as the medical equipments

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This equipment has been tested and found to comply with the limits for medical devices in IEC 60601-1-2. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation

#### Power Cord

For use of equipment in rated voltage less than 125Vac, minimum 6A, Type SJT or SVT, 18/3AWG, 10A, max 3.0m long: One end with Hospital Grade Type, NEMA 5-15P other end with appliance coupler. For use of equipment in rated voltage less than 250Vac, minimum 6A, Type SJT or SVT, 18/3AWG, 10A, max 3.0m long: One end terminated with blade attachment plug (HAR) Type, NEMA 6-15P.

This product may malfunction due to electromagnetic waves caused by portable personal telephones, transceivers, radio-controlled toys, etc. Be sure to avoid having objects such as, which affect this product, brought near the product.

The information in this publication has been carefully checked and is believed to be entirely accurate at the time of publication. HUVITZ assumes no responsibility, however, for possible errors or omissions, or for any consequences resulting from the use of the information contained herein.

HUVITZ reserves the right to make changes in its products or product specifications at any time and without prior notice, and is not required to update this documentation to reflect such changes.

9000J200002-I

(2025.08)

ANVISA : 81325990375

---

©2009 HUVITZ Co., Ltd.

38, Burim-ro 170beon-gil, Dongan-gu, Anyang-si, Gyeonggi-do, 14055

435-862, Republic of Korea

All rights are reserved.

Under copyright laws, this manual may not be copied, in whole or in part, without the prior written consent of HUVITZ Co., Ltd.

## Contents

|                                                    |           |
|----------------------------------------------------|-----------|
| <b>1. Introduction .....</b>                       | <b>6</b>  |
| 1.1. Indication for use .....                      | 6         |
| 1.2. Classification .....                          | 6         |
| <b>2. Safety Information .....</b>                 | <b>7</b>  |
| 2.1. Introduction.....                             | 7         |
| 2.2. Safety Symbols.....                           | 8         |
| 2.3. Environment factors .....                     | 13        |
| 2.4. Safety Precautions .....                      | 15        |
| <b>3. Features &amp; Operating Condition .....</b> | <b>20</b> |
| 3.1. Brief Description .....                       | 20        |
| 3.2. Operating Condition.....                      | 20        |
| <b>4. Standard Components .....</b>                | <b>23</b> |
| <b>5. Accessories .....</b>                        | <b>24</b> |
| <b>6. Composition.....</b>                         | <b>25</b> |
| <b>7. Installation .....</b>                       | <b>31</b> |
| 7.1. Assembling the Instrument.....                | 31        |
| <b>8. Trial Operations .....</b>                   | <b>35</b> |
| <b>9. Main Features .....</b>                      | <b>36</b> |
| <b>10. Ordinary Maintenance .....</b>              | <b>37</b> |

|            |                                           |           |
|------------|-------------------------------------------|-----------|
| 10.1.      | Halogen Projection bulb or LED lamp ..... | 38        |
| 10.2.      | Protection fuses replacement.....         | 40        |
| 10.3.      | Cleaning & Disinfection.....              | 41        |
| 10.4.      | Disposal .....                            | 41        |
| <b>11.</b> | <b>EMC Information.....</b>               | <b>42</b> |
| <b>12.</b> | <b>Electric Diagram .....</b>             | <b>44</b> |
| <b>13.</b> | <b>Technical Specifications .....</b>     | <b>45</b> |
| <b>14.</b> | <b>Service Information .....</b>          | <b>47</b> |

## 1. Introduction

### 1.1. Indication for use (intended use)

The SLIT LAMP, HS-7000 is intended for use in eye examination of the anterior eye segments, from the cornea epithelium to the posterior capsule. It is used to aid in the diagnosis of diseases or trauma which affects the structural properties of the anterior eye segment.

### 1.2. Classification

- Protection against electric shock: Class I
- Applied Parts: Type B applied parts
- Protection against harmful ingress of water: Ordinary, IPX0
- Degree of safety in the presence of a flammable anesthetics mixture with air or with oxygen or with nitrous oxide: Not suitable for use in the presence of a flammable anesthetics mixture with air or with oxygen or with nitrous oxide.
- Mode of operation: Continuous
- Light Hazard protection Classification: Group 1 by ISO15004-2:2007
- Stationary: Do not try to move this equipment without help. This microscope is heavy and should maintain stationary. Always get two persons to help lift this unit
- Classification of product: Class I according to the MDR(EU/2017/745)

**CAUTION**

Federal law restricts this device to sale by or on the order of a physician

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## 2. Safety Information

### 2.1. Introduction

Safety is everyone's responsibility. The safe use of this equipment is largely dependent upon the installer, user, operator, and maintainer. It is imperative that personnel study and become familiar with this entire manual before attempting to install use, clean, service or adjust this equipment and any associated accessories. It is paramount that the instructions contained in this manual are fully understood and followed to enhance safety to the patient and the user/operator. It is for this reason that the following safety notices have been placed appropriately within the text of this manual to highlight safety related information or information requiring special emphasis. All users, operators, and maintainers must be familiar with and pay particular attention to all Warnings and Cautions incorporated herein.

#### WARNING

“Warning” indicates the presence of a hazard that could result in severe personal injury, death or substantial property damage if ignored.

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#### NOTE

“Note” describes information for the installation, operation, or maintenance of which is important but hazard related if ignored.

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#### CAUTION

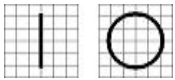
“Caution” indicates the presence of a hazard that could result in minor injury, or property damaged if ignored.

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
## 2.2. Safety Symbols



The International Electro technical Commission (IEC) has established a set of symbols for medical electronic equipment which classify a connection or warn of any potential hazards. The classifications and symbols are shown below.

This applies only to the instrument that has the certification symbol printed explicitly on the product label or sticker.

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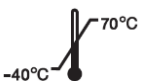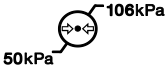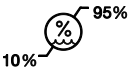## 2.3. Environment factors

Avoid the following environments for operation or storage:

Where the instrument is exposed to water vapor.  
Don't operate the instrument with wet hands Indoor use only.

---

Where the instrument is exposed to direct sunlight.

---

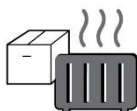
A place where the equipment can be exposed to direct ultraviolet.

---

Where there are big changes in temperature.  
Optimal temperature range for normal operation is from 10°C to 40°C (Humidity : 30 ~ 90%).

---


Where there is hot equipment nearby.

---

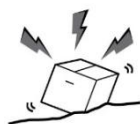Where the humidity is extremely high or there is a ventilation problem.

Where the instrument is exposed to excessive shocks or vibrations.

---

Where the instrument is exposed to chemical material or explosive gas.

---

Be cautious so that things like dust and metal do not fall inside the instrument.

---

Don't disassemble or open the product. HUVITZ does not take responsibility for the possible problems

---

Be careful not to block the fan of the instrument.

---

Don't plug the AC power cable into the outlet unless all parts of the instrument are completely connected. Otherwise, it will cause severe damage on the instrument.

---

Pull out the power cable with holding the plug, not the cord.  
To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

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This equipment has been developed and tested according to safety standards as well as national and international standards. This guarantees a very high degree of safety for this device. The legislator expects us to inform the user expressively about the safety aspects in dealing with the device. The correct handling of this equipment is imperative for its safe operation. Therefore, please read carefully all instructions before switching on this device. For more detailed information, please contact our Customer Service Department or one of our authorized representatives.

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1. This equipment must not be used (a) in an area that is in danger of explosions and (b) in the presence of flammable, explosive, or volatile solvent such as alcohol, benzene or similar chemicals.

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2. Do not put or use this device in humid rooms. Humidity should be maintained between 30% and 90% for normal operation. Do not expose the device to water splashes, dripping water, or sprayed water. Do not place containers containing fluids, liquids, or gases on top of any electrical equipment or devices.

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3. The equipment must be operated only by, or under direct supervision of a properly trained and qualified person.

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4. Modifications of this equipment may only be carried out by Huvitz's service technicians or other authorized persons.

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5. Customer maintenance of this equipment may only be performed as stated in the User's Manual and Service Manual. Any additional maintenance may only be performed by Huvitz's service technicians or other authorized persons.

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6. The manufacturer is only responsible for effects on safety, reliability, and performance of this equipment when the following requirements are fulfilled: (1) The electrical installation in the respective room corresponds to the specifications stated in this manual and (2) This equipment is used, operated, and maintained according to this manual and Service Manual.

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7. The manufacturer is not liable for damage caused by unauthorized tampering with the device(s). Such tampering will forfeit any rights to claim under warranty.

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8. This equipment may only be used together with accessories supplied by Huvitz's. If the customer makes use of other accessories, use them only if their safe usability under technical safety aspects has been proved and confirmed by Huvitz or the manufacturer of the accessory.

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9. Only persons who have undergone proper training and instructions are authorized to install, use, operate, and maintain this equipment.

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10. Keep the User's Manual and Service Manual in a place easily accessible at all times for persons operating and maintaining the equipment.

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11. Do not force cable connections. If a cable does not connect easily, be sure that the connector (plug) is appropriate for the receptacle (socket). If you cause any damage to a cable connector(s) or receptacle(s), let the damage(s) be repaired by an authorized service technician.

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12. Please do not pull on any cable. Always hold on to the plug when disconnecting cables.

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13. Before every operation, visually check the equipment for exterior mechanical damage(s) and for proper function.

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14. Do not cover any ventilation grids or slits.

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15. Immediately turn off and unplug any equipment that gives off smoke, sparks, strange noises, or odors.

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16. Power Supply Cord set (international) - (Detachable) Rated 10 A, 250 V. Plug type CEE 7/VII, Connector type IEC 60320/C13, and Cord type H05VV-F3G, min. 0.75 mm<sup>2</sup>, 3-conductor terminating in molded-on grounding type attachment plug. <HAR> marked on the cord. Maximum 4.0 m long.

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### 3. Features & Operating Condition

#### 3.1. Brief Description

The device works like all slit lamps:

- Stereomicroscopic examination of the eye under the slit light.
- Microscopic examination of the ocular fundus and posterior vitreous body (using hruby's lens).

These devices are designed for use by ophthalmologists and optometrists (within the realms of their respective professions) for specific diagnostic procedures (bio-microscopic examination of the eye)

#### 3.2. Operating Condition

This instrument must be followed by these following conditions:

##### -. Operation

An ambient temperature range of 10°C ~ 40°C (50°F ~ 104°F)

A relative humidity range of 30% ~ 90%(with non-condensing)

An atmospheric pressure range of 800 ~ 1060hpa

An indoor use, Altitude up to 2000 m

##### -. Transportation(with packaging)

An ambient temperature range of -40°C ~ 70°C (-40°F ~ 158°F)

A relative humidity range of 10% ~ 95%

An atmospheric pressure range of 500 ~ 1060hpa

##### -. Storage

An ambient temperature range of -10°C ~ 55°C (14°F ~ 131°F)

A relative humidity range of 10% ~ 95% (with non-condensing)

An atmospheric pressure range of 700 ~ 1060hpa

Please avoid where the equipment is exposed to excessive shocks or vibration.

The device neither generates nor receives electromagnetic interference when used near other equipment. Therefore, no preventive or corrective measures are required

Because prolonged intense light exposure can damage the retina, the use of the device for ocular examination should not be unnecessarily prolonged, and the brightness setting should not exceed what is needed to provide clear visualization of the target structures. This device should be used with filters that eliminate UV radiation ( $<400\text{nm}$ ) and, whenever possible, filters that eliminate short-wavelength blue light ( $<420\text{nm}$ ).

The retinal exposure dose for a photochemical hazard is a product of the radiance and the exposure time. If the value of radiance were reduced in half, twice the time would be needed to reach the maximum exposure limit.

While no acute optical radiation hazards have been identified for slit lamps, it is recommended that the intensity of light directed into the patient's eye be limited to the minimum level which is necessary for diagnosis. Infants, aphakes and persons with diseased eyes will be at greater risk. The risk may also increased if the person being examined has had any exposure with the same instrument or any other ophthalmic instrument using a visible light source during the previous 24 hours. This will apply particularly if the eye has been exposed to retinal photography

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## 4. Standard Components

The instrument is delivered in package. Taking it out of the package, make sure that none of the following parts is missing.

- A table top(11)(N.B. the table top isn't supplied with the slit lamp for twin table) on which are placed: A SMPS box(15) including the main switch & a warning light (13), Two sliding guides for the base with orthogonal movements(17), A sliding plate for the position device(19), A drawer(16);

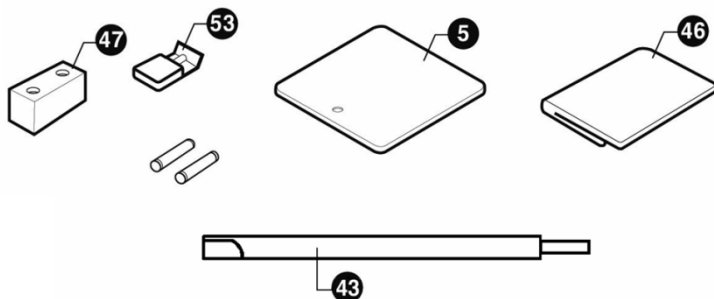
If Unit Table (61) is selected as an option, a metal table will be provided instead of the standard top plate (11).

- A base with orthogonal movements.
- A microscope with either dual magnification or a system (31) including eyepieces.
- An optical group of the slit projector.
- A head rest module (59).
- A microscope arm (45).
- This manual.
- A set of accessories including; two wheels for the sliding guides(18); a setting rod(43); a protective cover(46); a spare bulb 12V 30W Halogen bulb(53) – Except LED model; two protection fuses(48); a breath screen(19); a chin-rest paper(47);a hole cover(45); a power cable; two patient's handles(50)

## 5. Accessories

The following standard accessories are provided with the instrument;

- protection cover(46)
- a setting rod(43)
- spare bulbs  
12V 30W Halogen bulb (53) – Except LED model
- chin-rest paper(47)
- spare fuses  
250Vac T2.0AL
- a breath screen(5)
- power cable

[ Figure 1. Accessories ]

## 6. Composition


- 1) Fixation point
- 2) Control lever for changing enlargement
- 3) Removable eye-pieces
- 4) Knob for clamping the breath screen
- 5) Breath screen
- 6) Control handle for the slit rotation 90-0-90
- 7) Connecting plug of the projector lamp
- 8) Knurled ring for filters insertion
- 9) Joystick for lateral, longitudinal and vertical movements(x y z)
- 10) Base with orthogonal movements
- 11) Shaped tabletop
- 12) Warning light indicating the ignition of the SMPS is on
- 13) Main switch & Warning light
- 14) Luminosity selector
- 15) SMPS box
- 16) Drawer
- 17) Toothed guides
- 18) Wheel protection crankcases
- 19) Sliding plate
- 20) Knob to lock the projector
- 21) Scale for the projector position
- 22) Knob to lock the arm of the microscope
- 23) Hole cover
- 24) Knurled knob to lock the Lamp housing
- 25) Control handle for the vertical setting of the chin-rest
- 26) Locking pins for chin rest paper

- 27) Chin rest
- 28) Knobs to change the slit width
- 29) Pointer for the eye positioning
- 30) Forehead Rest
- 31) Microscope
- 32) Lamp socket
- 33) Knob to lock the base of the instrument
- 34) Knob to lock the microscope
- 35) Control handle for horizontal tilting
- 36) Control lever for vertical tilting
- 37) Microscope position stop
- 38) Graduated ring 90-0-90 determining the inclination of the slit during rotation
- 39) Lamp cover
- 40) fuse slots
- 41) Connecting plug
- 42) Fixation point connector
- 43) Setting rod
- 44) Microscope separation knob
- 45) Microscope arm
- 46) Protection cover
- 47) Chin rest papers
- 48) Guide screw
- 49) An optical group of the slit projector (Halogen or LED)
- 50) Patient's handle
- 51) Head rest fastening screw
- 52) Wheel
- 53) Halogen bulb 12V 30W
- 54) Bulb fastening knob


- 55) Mirror
- 56) Capture connector (For the image system)  
Or Rubber cap (No image system)
- 57) Capture switch
- 58) Luminosity selector connector ( Connect the Luminosity selector to the SMPS)
- 59) Head rest module
- 60) LED Lamp (Option)
- 61) Unit table (Option)

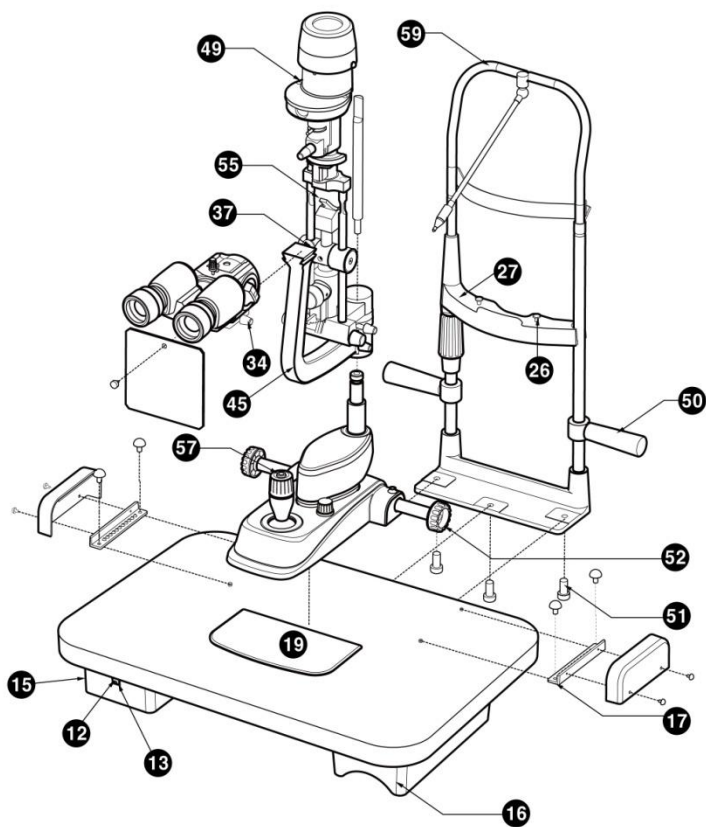[ Figure 2. Composition I ]

[ Figure 3. Composition II ]

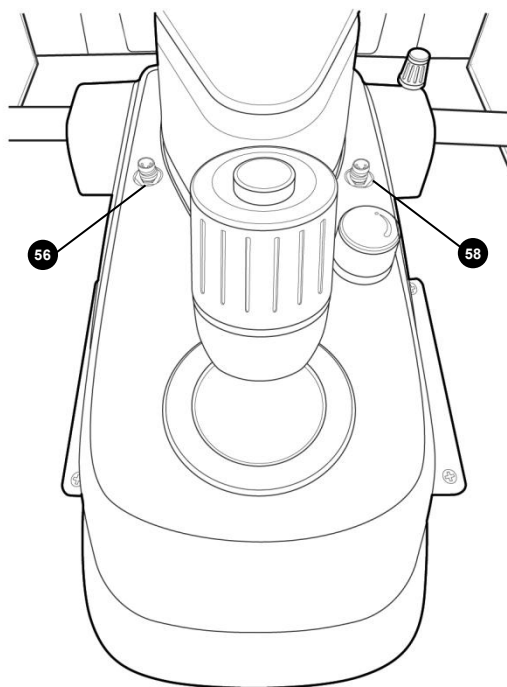[ Figure 4. Composition III ]


## 7. Installation
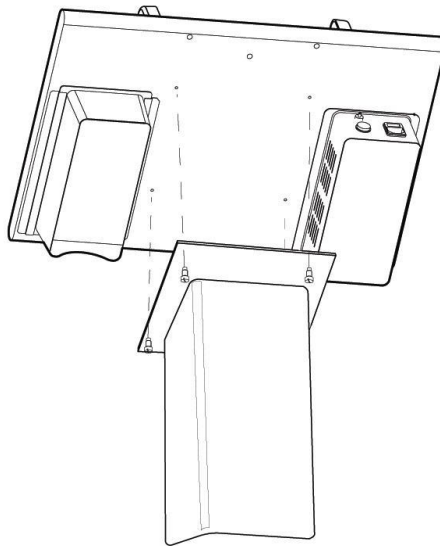
### NOTE

The appliances are always supplied with excellent packaging to be able to stand up to all kinds of transport and standard storage conditions. Please contact the suppliers if any irregularities are found in the appliance due to transport after unpacking


### 7.1. Assembling the Instrument

- Put the table top on a very stable support. Block the two parts together by tightening the screws

[ Figure 5. Table Assembling ]

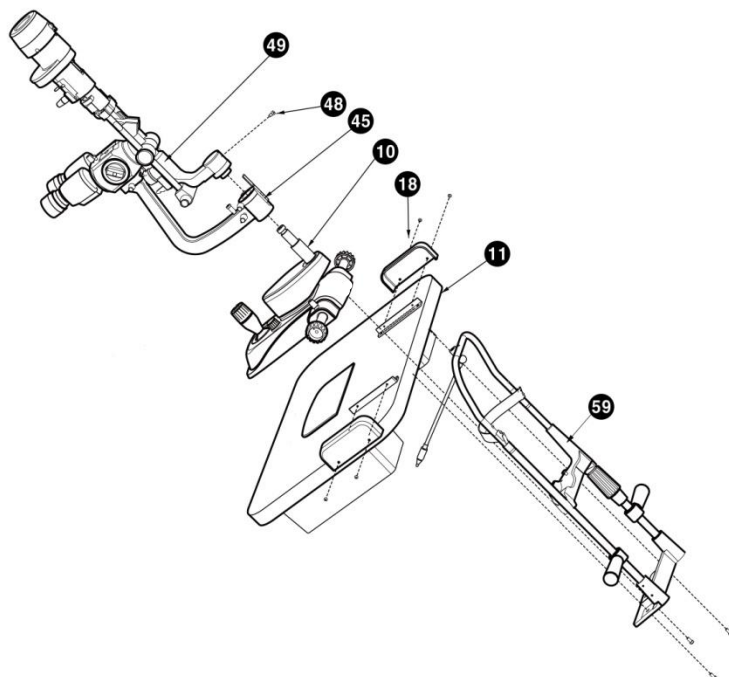
- Place the head rest module (59) and make sure that its holes fit together with those of the table top. Tighten the screw with the screw driver


[ Figure 6. Assembling the instrument I ]

- Place the base with orthogonal movements on the guides situated on the upper part of the table top, checking that the wheels are in line. Then Place the crankcase (59) and make sure that its holes fit together with those of the toothed guides (17). Tighten the screw with the screw driver(+)
- Assemble the microscope arm (45) on the Base with orthogonal move ments (10).

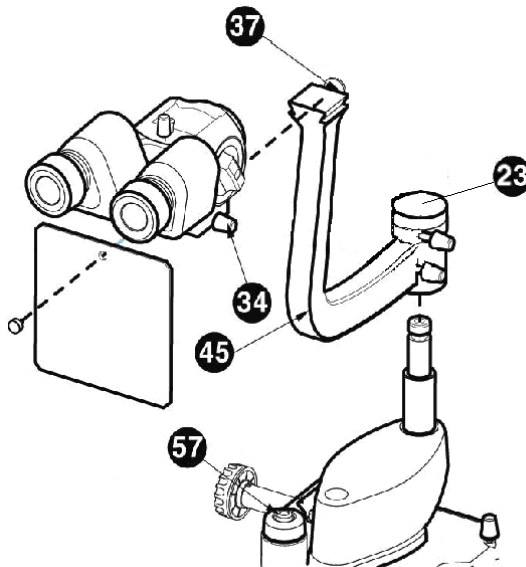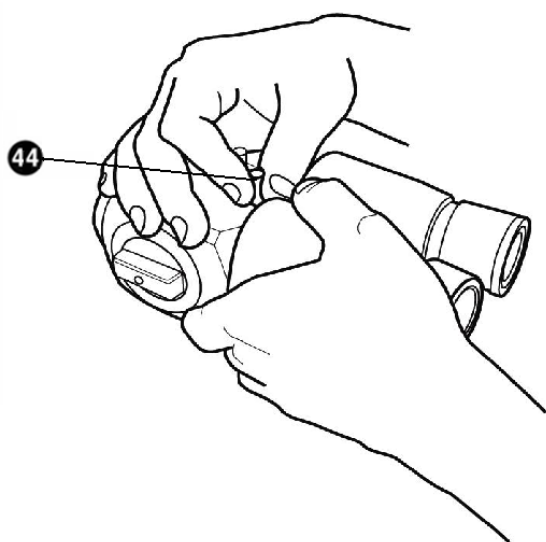- Assemble an optical group of the slit projector (49) on the Base with orthogonal movements (10). Then tighten the Guide screw with the screw driver(-)
- Put the microscope(31) in the right position making sure that it is placed well in the stop(37); then tighten it with the knob(34)situated the right side of the microscope.
- Fasten the breath screen(5) with the knob(4)
- Insert the connecting cable into the socket.

[ Figure 7. Assembling the instrument II ]

## 8. Trial Operations

- Assemble the chin-rest paper (47) on the chin-rest (27).
- Let the patient sit down
- comfortable with his chin on the chin-rest and his forehead against the forehead-rest.
- Lift or lower the chin-rest by moving the handle (25) so that the patient's eyes are in line with the notches on the chin-rest(29)
- Switch the instruments on, press the switch (13). You will see the warning light on (12).
- Adjust the luminous intensity by moving the selector(14)
- Frame and focus the eye to be examined by moving the lever(9)

## 9. Main Features

Principal operative characteristics of the instrument:

- Observation without settling or tiring and rapid change of enlargements.
- Base with cross-wise movement (which can be stopped) commanded by a single joystick for all adjustments(x y z)
- Stereomicroscopic observation of the eye with enlargements 6X-10X-16X-25X-40X (5 enlargements model).
- Tilting 5°-10°-15°-20°
- Microscopy of the back part and of the posterior vitreous body by means of the Hruby lens or a contact options. (Goldman lens).
- Observation of the eye and valuation of the position of contact lenses in a fluorescent light.
- Tonometry with Goldman tonometer.
- Measurement of the cornea diameter and contact lenses.
- Survey which reveals the axis of the contact lens, thanks to the possible reading of the rotation angle of the slit which includes all the meridians.

## 10. Ordinary Maintenance

### WARNING

The repairs illustrated below should all be carried out with the power cable disconnected from the main supply plug. Please contact the suppliers for any other problems that cannot be resolved by the operations described below.

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### CAUTION

Please turn off the power switch before replacing lamps. Please place lamps after cooling down. The heated lamp may burn your fingers.

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### NOTE

Don't touch any other parts of glass tube with the bare fingers. Fingerprints, if any, must be wiped off from the glass tube with alcohol

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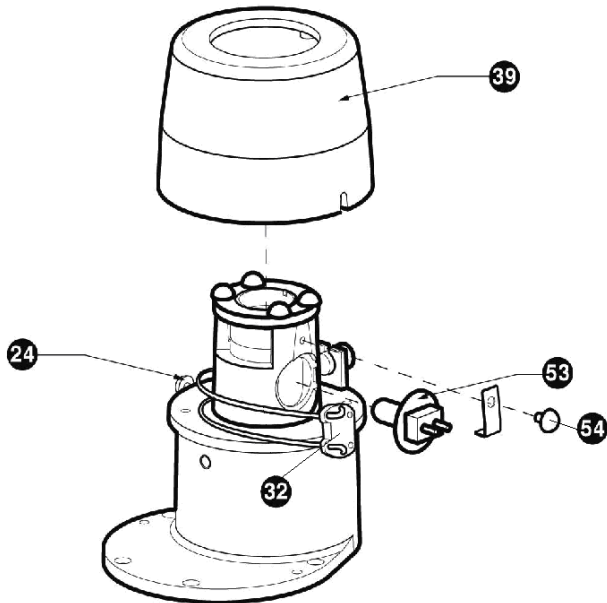
### 10.1. Halogen Projection bulb or LED lamp

- Please contact the suppliers for any LED lamp problems.

To replace the Halogen projection bulb proceed as follow.

- Before doing anything else, disconnect the plug from the main supply socket;
- Remove the screws with the driver(24), Remove the cover(39),
- Remove the Lamp socket(32) from the bulb stems;
- Bulb fastening knob (54) to loosen the bulb stop
- Remove the burnt out bulb; be careful , it could be very hot;
- Replace it with a new one(12V30W Halogen) using the plastic envelope(never touch the bulb whit fingers)
- Fasten the bulb with the knob
- Make sure that the stems of the bulb are completely inserted; insert the plug in the main supply socket

[ Figure 8. Replace Halogen bulb ]

 **WARNING**

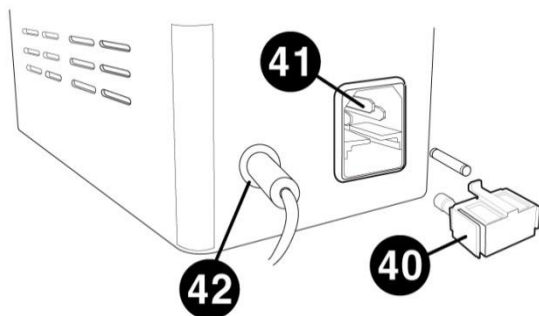
Don't Touch to the Lamp House, During the Operation. It's Heating up.

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## 10.2. Protection fuses replacement

To replace the fuses proceed as follows:

- The protection fuses are placed on the back of the SMPS, and inserted power socket. Before doing anything else, disconnect the plug from the main supply socket
- Remove the burned-out fuses
- Substitute the fuses and check
- Replace the plug in the main supply socket.

[ Figure 9. Replace the fuse ]

### CAUTION

Replace fuses with the same rating and type(250Vac, T2.0AL)

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### 10.3. Cleaning & Disinfection

- To keep the device clean, apply clinical alcohol and cidex or cidex plus on a soft cloth and scrub the body and the probe once a month. Do not use lacquer, thinner, ethylene, or the oxidizing substance.
- If the instrument is not in use, dust must be kept away by covering the instrument with the supplied plastic cover. Dust accumulating during the functioning of the instrument must be periodically removed with a very soft cloth or with a small pump.

### 10.4. Disposal

- To dispose the instrument, accessories, and components, follow local governing ordinances and recycling plans regarding disposal or recycling of instrument or device components.
- When disposing packing materials, sort them by the materials and follow local governing ordinances and recycling plans.

## 11. EMC Information

Manufacturer announcement – electromagnetic waves trouble

| <b>Electromagnetic waves trouble</b>                                                                                                                                            |                                    |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------|
| HS-7000 should be used in the below mentioned electromagnetic wave environment. HS-7000 purchaser or user needs to confirm whether HS-7000 is used in this type of environment. |                                    |
| <b>Trouble test</b>                                                                                                                                                             | <b>Question of appropriateness</b> |
| RF emissions CISPR 11                                                                                                                                                           | Group 1                            |
| RF emissions CISPR 11                                                                                                                                                           | Class B                            |
| Harmonic emissions IEC 61000-3-2                                                                                                                                                | Class B                            |
| Voltage fluctuations/flicker IEC 61000-3-3                                                                                                                                      | Complies                           |

Manufacturer announcement – electromagnetic waves tolerance

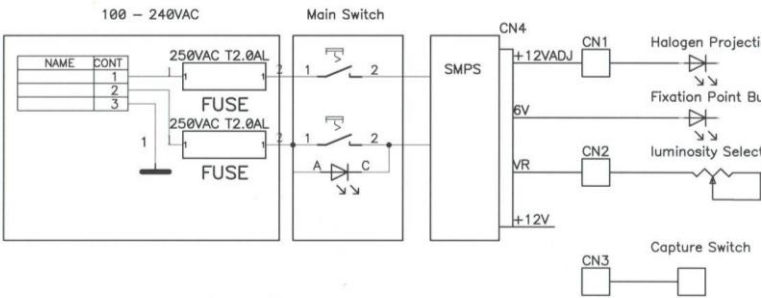
| <b>electromagnetic waves tolerance</b>                                                                                                                                                 |                                                                                                             |                                                                                                              |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------|
| HS-7000 is to be used in the below designated electromagnetic wave environment. HS-7000 customer and user need to guarantee that the HS-7000 will be used in this type of environment. |                                                                                                             |                                                                                                              |
| <b>Tolerance test</b>                                                                                                                                                                  | <b>IEC 60601 test level</b>                                                                                 | <b>Appropriateness level</b>                                                                                 |
| Electrostatic discharge(ESD)<br>IEC 61000 - 4 - 2                                                                                                                                      | contact $\pm 8$ kV<br>in the air $\pm 15$ kV                                                                | contact $\pm 8$ kV<br>in the air $\pm 15$ kV                                                                 |
| Electric rapid transients/burst<br>IEC 61000 - 4 - 4                                                                                                                                   | power supplying line $\pm 2$ kV<br>input/output line $\pm 1$ kV                                             | power supplying line $\pm 2$ kV<br>input/output line $\pm 1$ kV                                              |
| Surge<br>IEC 61000 - 4 - 5                                                                                                                                                             | between lines $\pm 1$ kV<br>between line and grounding $\pm 2$ kV                                           | differential mode $\pm 1$ kV<br>common mode $\pm 2$ kV                                                       |
| Voltage dip, instantaneous interruption, voltage fluctuation at the power input line                                                                                                   | For 0.5 cycle $< 5\% UT(UT's > 95\% \text{ decrease})$<br>For 5 cycle $40\% UT(UT's 60\% \text{ decrease})$ | For 0.5 cycle $< 5\% UT(UT's > 95\% \text{ decrease})$<br>For 5 cycle, $40\% UT(UT's 60\% \text{ decrease})$ |

|                                                                   |                                                                                                                                  |                                                                                                                                    |
|-------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------|
| IEC 61000 - 4 - 11                                                | For 25 cycle<br>70 % <i>UT</i> ( <i>UT</i> 's 30 % decrease)<br>For 5 seconds < 5 %<br><i>UT</i> ( <i>UT</i> 's > 95 % decrease) | For 25 cycle,<br>70 % <i>UT</i> ( <i>UT</i> 's 30 % decrease)<br>For 5 seconds, < 5 %<br><i>UT</i> ( <i>UT</i> 's > 95 % decrease) |
| Power frequency magnetic field<br>(50/60 Hz)<br>IEC 61000 - 4 - 8 | 30 A/m                                                                                                                           | 30 A/m                                                                                                                             |

Other *UT* is the a.c. power voltage for before approving the test level.

| Electromagnetic waves tolerance                                                                                                                                       |                                                                                                                                                                |                       |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------|
| HS-7000 is to be used in the below mentioned electromagnetic wave environment. HS-7000 purchaser or user needs to confirm whether HS-7000 issued at this environment. |                                                                                                                                                                |                       |
| Tolerance test                                                                                                                                                        | IEC 60601 test conditions                                                                                                                                      | Appropriateness level |
| Conductivity RF electromagnetic field<br>IEC 61000 - 4 - 6                                                                                                            | 3 Vrms<br>150 kHz~80 MHz<br><br>6 Vrms in ISM band<br>6.765 MHz to 6.795 MHz<br>13.553 MHz to 13.567 MHz<br>26.957 MHz to 27.283 MHz<br>40.66 MHz to 40.70 MHz | 3 Vrms<br><br>6 Vrms  |
| Radioactivity RF electromagnetic field tolerance<br>IEC 61000 - 4 - 3                                                                                                 | 3 V/m<br>80 MHz~2.7 GHz scope                                                                                                                                  | 3 V/m                 |

12. Electric Diagram

### 13. Technical Specifications

| MICROSCOPE - HS-7000 |                                                                                   |     |       |     |     |                |
|----------------------|-----------------------------------------------------------------------------------|-----|-------|-----|-----|----------------|
| Model name           | Halogen: HS-7000(HLG)<br>LED : HS-7000(LED)                                       |     |       |     |     |                |
| Type                 | Galilean converging binocular                                                     |     |       |     |     |                |
| Magnification        | 5 positions rotation drum                                                         |     |       |     |     |                |
| Eyepiece             | 12.5x (Option 10x)                                                                |     |       |     |     |                |
| Total magnification  | 6x                                                                                | 10x | 16x   | 25x | 40x | Drum Mark      |
|                      | 6x                                                                                | 10x | 16x   | 25x | 40x | 12.5x Eyepiece |
|                      | 4.8x                                                                              | 8x  | 12.8x | 20x | 32x | 10x Eyepiece   |
| Field of view        | 38.5, 24.0, 15.0, 9.0, 6.0 mm                                                     |     |       |     |     |                |
| Pupil Distance       | 55 ~ 80 mm                                                                        |     |       |     |     |                |
| SLIT ILLUMINATION    |                                                                                   |     |       |     |     |                |
| Slit length          | 0.3 ~ 14 mm                                                                       |     |       |     |     |                |
| Slit width           | 0 ~ 14 mm                                                                         |     |       |     |     |                |
| Slit projection      | 1x                                                                                |     |       |     |     |                |
| Aperture diaphragms  | 0.3/1/3/5/9/14 mm                                                                 |     |       |     |     |                |
| Filters              | Cobalt blue, Red-free (Green filter),<br>Gray and Heat- absorption, Yellow filter |     |       |     |     |                |
| Slit rotation        | 0°~180°                                                                           |     |       |     |     |                |
| Angle of light       | 0°, 5°, 10°, 15°, 20°                                                             |     |       |     |     |                |

| BASE                  |                                                                                |
|-----------------------|--------------------------------------------------------------------------------|
| Vertical Movement     | 30mm                                                                           |
| Longitudinal Movement | 100mm.                                                                         |
| Lateral Movement      | 100mm                                                                          |
| Power supply          | AC 100-240V, 50/60Hz, 0.8-0.5A(Halogen)<br>AC 100-240V, 50/60Hz, 0.3-0.2A(LED) |

## 14. Service Information

How to contact service: If there are any problems with the equipment, please follow the steps below:

- First of all, refer to the 10. Maintenance and 11. Troubleshooting sections according to the problem that you are encountered. And then follow the suggested sequences.
- If the problem persists, please contact the local distributor in your province or country at first.
- Before calling to the local distributor, you'd better check these information such as Model and Serial Numbers. To do so, fill up the following table as soon as you purchase our product. You can look up these information at any time. The serial number is found on the back of this unit. The serial number is unique to this unit. You should retain this manual as a permanent record of your purchase. Please retain your purchase receipt as your proof of purchase.

**Date of**

**Purchase:**

**Dealer's Name:**

**Dealer Address:**

**Dealer Phone**

**No. :**

**Model No.:**

**Serial No.:**

- If you can't contact with your local distributor, you can directly get in touch with the service department of the HUVITZ using the phone number and the address written in the below table.